



1. Place collector mouth piece to lips and begin collecting specimen.
2. Rotate or tip the device to collect an equal amount of saliva in both tubes.

3. Continue Steps 1 and 2 while tilting device from side to side until both tubes have reached minimum volume.

4. Carefully remove the purple mouth piece by placing the thumb under the lip of the collector and pushing up – Keep tubes upright to prevent spilling samples.

5. A) Place purple caps firmly onto top of collection tubes.
B) Place a Tamper Evident Seal on each capped tube. Have Donor Initial and Date both tubes.

Important: At the end of the saliva collection process there should be two (2) capped tubes, each with a tamper evident seal, initialed and dated.



For additional information, please go to our website at www.sciteck.org
 Contact: 800-905-4272 / 828-650-0409
info@sciteck.org

© 2011 Sciteck, Inc. All Rights Reserved